

easy
safe

The Easy & Safe locking mechanism can be fitted/removed in seconds and consists of three components:

Rear anchoring system

The rear part is at a slight angle and has a security lip. This particular design detail allows the Easy & Safe seats to fit even on those toilets with a wide, rear internal rim.

Lateral guides

The two lateral guides allow the Easy & safe seats to be fitted correctly and offer greater stability.

Front-locking clamp

Turning the screw clockwise on the front locking device securely clamps the seat to the toilet. The clamp is covered in an anti-slip rubber and is 7cm / 2³/₄" wide guaranteeing greater grip.

Four anti-slip rubber strips are located on the underside of the seat for extra stability.

an easy and safe world

Raised Toilet Seat
made in italy



The Easy & Safe raised toilet seat features a front-locking mechanism for easy installation and has a rear anchoring system for maximum stability.

The seat's surface is gently contoured for comfort and is designed to give better positioning of the pelvis. The slight forward incline of the seat reduces flexion of the hip and knee joints and assists the user when lowering and raising.

ES-11-10
Height 10cm / 4"
ES-11-14
Height 14cm / 5¹/₂"

ES-5-10
Height 10cm / 4"
ES-5-14
Height 14cm / 5¹/₂"



Ergonomically shaped raised toilet seat with arm-rests and "Easy & Safe" locking system. The upper part of the aluminium armrest is covered with a soft anti-slip rubber. The Easy & Safe "ES11" model is fitted with padded arm-rests offering the user even greater support and security.


Ergonomically shaped raised toilet seat with "Easy & Safe" locking system.

The "Easy & Safe" raised toilet seat is made of white, HDPE plastic. The front-locking clamp is made of fibre-glass offering strong resistance.



Maximum user weight 160 kg / 350 lb


an easy and safe world

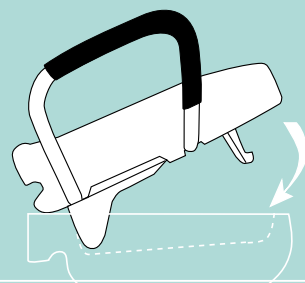

1


The Easy & Safe raised toilet seat can be installed and removed in seconds:

Before fitting the Easy & Safe raised toilet seat, the screw must be fully turned anti-clockwise.

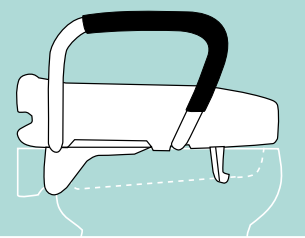

2

Position the seat by first inserting the rear part inside the rear toilet rim and then lowering the front. To lock, turn the screw clockwise until the locking system clamps onto the inside rim of the toilet bowl and a resistance is felt.

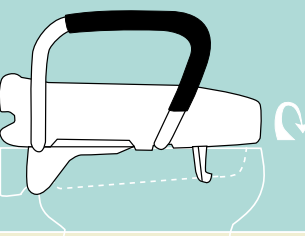

3

To remove the seat, turn the screw anti-clockwise and lift away from the front.

CE

Clean with damp sponge and if necessary with a neutral cleanser. Do not soak the seat in water.

Latex Free



Raised Toilet Seat

Alzawater
Klosettsitzerhöhung
Rehausse toilettes
Elevador de inodoro
Ανυψωτικό τουαλέτας
Alteador para sanita
Toiletsædeforhøjning
Portabel Toaförhöjning
Forhøyet toalettsete
Nakzadka toaletowa

Manufactured by

RehabInnovation

Via Privata OTO, 23
19136 La Spezia
Italy
ph. +39 0187 575104
fax +39 0187 517706
www.rehabinnovation.com
e-mail: info@rehabinnovation.com

